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**ENGLISH**

**INTENDED USE**  
 Kit intended to determination of calcium in serum, plasma and urine. Diagnostic use only.

**STORAGE AND HANDLING**

- Store at 15 to 30 °C.
- Protect from light.
- Stable until the kit expiration date that is printed on the label.
- Do not use reagents whose date has expired.

**WORKING PRINCIPLE**

Calcium forms with Arsenazo III a complex of blue color that can be measured at 650-660 nm. The intensity of the formed color is proportional to the amount of calcium present in the sample.

**SAMPLE – PREPARATION AND STABILITY**

**Sample Type:** Serum, plasma (Heparin) and urine.  
**Collection, handling and preparation:** Make the collection of the sample in accordance with Good Laboratory Practice. The samples to be analyzed should be treated as potentially infectious material.  
**Preservation:** The calcium in serum and plasma (heparina) is stable for 14 days if stored at a temperature of 4 to 8 °C and 6 months if stored at a temperature of -20 °C. Urine: Use sample collected within 24 hours in a bottle containing 20 to 30 mL of 6 mol / L HCl solution to prevent the precipitation of calcium salts. Mix well, measure the volume and proceed with the test.

**PRODUCT DESCRIPTION**

<b>R 1</b>	MES buffer ≥ 50 mmol/L; Arsenazo III ≥ 0,1 mmol/L
<b>STD</b>	Preservative; calcium chloride concentration equivalent to 10mg/dL

The determination of calcium is traceable to reference material NIST 3109 a.

**STABILITY IN USE**

- The stability of the product (R1 and STD) in use is 24 months as long as followed by the recommended storage conditions (15 to 30 ° C).

**TECHNICAL PROCEDURE**

**A) REAGENT PREPARATION**  
**Reagent 1 (R1)**  
 The reagent is ready to use.  
**B) OPERATING RANGE**  
 The product operating range is from 0,94 mg/dL to 20,00 mg/dL. For higher values, dilute the sample with NaCl 150 mM (0.9%), repeat the assay and multiply the result by the dilution factor.

**QUALITY CONTROL**

Use of controls should be a routine practice in the lab. It is suggested to use a control within the reference range or decision level and another control with value in another range of clinical significance. For the internal quality control of the lab it's indicated the use of the calibrator serum and control sera below:

Calibrator Serum - Autocal H	<b>REF</b>	13.002.00
Normal Control Serum – Quantinorm		13.003.00
Pathological Control Serum – QuantiAlt		13.004.00

**TEST PROCEDURE, CALCULATION AND INTERPRETATION**

**A) TEST PROCEDURE**  
 1. Pipette in the assay tubes:

	Blank	Standard	Sample
STD	-	10 µL	-
Sample	-	-	10 µL
R1	1,0 mL	1,0 mL	1,0 mL

2. Homogenize and incubate 2 minutes at 15 to 30 °C.
3. Measure the absorbance of Standard (Ast) and Sample (Asa) against the blank at 650-660 nm. The final reaction is stable for 20 min.

**B) CALCULATIONS**

**Calculation of serum**  
 Calcium (mg/dL) =  $\frac{\text{Sample Absorbance}}{\text{Standard Absorbance}} \times \text{STD Concentration}$

**Example:**  
 Standard Concentration = 10 mg / dL  
 Sample Absorbance = 0.320  
 Standard Absorbance = 0.350  
 Calcium (mg / dL) =  $\frac{0,320}{0,350} \times 10 = 9,1$  mg/dL

**With Calibration Factor (CF):**  
 CF =  $\frac{\text{STD Concentration (mg/dL)}}{\text{STD Absorbance}}$

Calcium (mg/dL) = Sample Absorbance x CF

**Example:**  
 Calibration Factor =  $\frac{10}{0,350} = 28,6$   
 Calcium (mg/dL) = 0,320 x 28,6 = 9,1 mg/dL

**24-hour urine:**  
 Calcium urine (mg/dL) =  $\frac{\text{Sample Absorbance}}{\text{Standard Absorbance}} \times \text{Standard Concentration (mg/dL)}$

Calcium urine (mg / 24 hours) =  $\frac{\text{Calcium urine (mg/dL)} \times \text{24 hour urine volume (mL)}}{100}$

**C) INTERPRETATION**

Calcium is the most predominant cation in the body. The skeleton contains 99% of body calcium with an unknown structure and composition roughly that of hydroxyapatite. The remainder of the calcium is distributed in tissues and extracellular fluid where it performs specific functions in vital processes, among them: blood coagulation, neuromuscular conduction, cardiac and skeletal muscle excitability, enzymatic activation, and preservation of cell membrane permeability and integrity. Serum calcium levels are controlled by parathyroid hormone (PTH), calcitonin, and vitamin D. An imbalance in some of these modulators causes changes in serum and body calcium levels. Serum increases in PTH or vitamin D are generally associated with hypercalcemia. Increased levels of serum calcium can be observed in multiple myeloma and other neoplastic diseases. Hypocalcemia can be observed for example in hypoparathyroidism, nephrosis and pancreatitis.

**INTERFERING AND LIMITATIONS**

**Hemolyzed, Jaundice and Lipemic Sera:** Hemoglobin > 200 mg/dL, Bilirubin > 15 mg/dL, Triglyceride > 900 mg/dL. Interfere with the dosage  
**Drugs:** see recommended reference (Young, 2000).

#### PERFORMANCE CHARACTERISTICS

**Sensitivity:** Detection limit: 0,45 mg/dL / quantification limit: 0,94 mg/dL  
**Analytical Specificity:** The product determines calcium specifically in the presence of other interfering substances in the sample, up to the concentrations stated above.  
**Accuracy:** The method was compared with a similar method for the determination of 40 samples in duplicate. The regression equation obtained was  $y = 0,988x + 0,081$  and the correlation coefficient  $r = 0,9979$ . Using this equation the total systematic error estimated is 0,42% to a level of 5,0 mg/dL and -0,69% to a level of 16,0 mg/dL.  
**Precision:** Determined using samples in 03 levels of decision, with two runs in duplicate per day during 20 days, being obtained.

Samples (mg/dL)	Repetitions	Within-Run Precision		Total Precision	
		SD	%CV	SD	%CV
8,207	80	0,036	0,4	0,037	0,5
12,220	80	0,054	0,4	0,054	0,4
17,445	80	0,068	0,4	0,074	0,4

% CV: Coefficient of variation expressed as a percentage; SD: Standard Deviation

#### RESIDUAL RISK, WARNINGS AND PRECAUTIONS

- Use Protective Equipment in accordance with Good Laboratory Practice.
- Do not mix reagents from different lots.
- Do not exchange the caps from different reagents in order to avoid cross contamination.
- Do not use the reagent if it displays any sign in disagreement with the one specified in the product MSDS.
- Use glass pipettes and disposable tips for each sample, control, standard / calibrator and reagent.
- The laboratory should establish chemical, microbiological and particulates requirements for the water prior to its use for each of its applications and must define the specifications or types of water that meets them. Once the required purity has been set, the purification system must be validated and it is important to ensure that the resulting water continues to meet specifications by means of periodic checks.
- The cleaning and drying of the lab material are fundamental factors for the stability of the reagents and reaching correct results.

#### REFERENCE RANGES

	mg/dL	mmol/L
Serum and Plasma	8,6 – 10,3	2,15 – 2,57
	mg/24 hours	mmol/24 hours
Urine	Up to 300	Up to 7,49

These values are intended for orientation only. It's recommended that each lab establishes its own reference range. Conversion for International System of Units (SI): mmol/L Calcium (mg/dL) X 0,25 = Calcium (mmol/L)

#### ADDITIONAL EQUIPMENT

- Spectrophotometer or photometer able to read at 650-660 nm.
- Pipettes and micropipettes.
- Clock or chronometer.
- Assay tubes.

#### WARNINGS AND PRECAUTIONS

- Discard the reactions surplus according to Good Laboratory Practice in a proper place for potentially infective material.
- The information for Disposing, Security and First Aid are described in the Manual Safety Data Sheet (MSDS) of this product available at [www.biotechnica.ind.br](http://www.biotechnica.ind.br) or calling for +55 35 3214 4646

#### QUALITY ASSURANCE / CUSTOMER TECHNICAL SERVICE

Before being approved for use, BioTécnica reagents are tested in the Quality Control Department. The quality of the reagents is assured up to the expiring date stated in the label of the external packaging, since it is stored and transported in the specified conditions. The quality control data concerning this product (batch printed on the labels of reagent bottles) or any technical doubt on handling this product or this procedure, contact us calling +55 35 3214 4646, your local distributor or sending an e-mail: [sac@biotechnicaltda.com.br](mailto:sac@biotechnicaltda.com.br)

#### AUTOMATION

This product is compatible to the most types of biochemical automatic analysers. The applications are available at [www.biotechnicaltda.ind.br](http://www.biotechnicaltda.ind.br)

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